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EAN: 5901098564007

Intended use:

- The organizer is intended for temporary storage and organization of medicines in solid form, such as tablets and capsules.

Safe use:

- This product is not a toy.
- The organizer is not child-protective. Keep out of the sight and reach of children and pets.
- Use only as intended.
- Do not use to store liquid medicines, loose powders, or chemicals unless they are intended for this type of storage.
- Before each use, check that the container and closure are clean, dry, and undamaged.
- Do not use the product if it is cracked, deformed, or does not close properly.
- To reduce the risk of mix-ups, do not place different medicines in one compartment, and keep dosage information together with the medicine's original packaging.
- If in doubt about the storage conditions of a medicine, follow the package leaflet or the advice of a pharmacist or doctor.

Cleaning and storage:

- Before first use and regularly thereafter, clean with warm water and a mild detergent; dry thoroughly after washing.
- Do not use strong chemicals or abrasive materials.
- Protect from high temperatures, open flame, and prolonged exposure to sunlight.

Additional information:

- Product identification details, the details of the responsible entity in the EU, the batch marking, and the country of origin should be checked on the product packaging or label.
- Dispose of the used or damaged product in accordance with local waste sorting rules.

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