POLSKI

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EAN: 5907998001079

- Intended use: The organizer is designed for orderly, short-term storage of tablets and capsules for 7 days, divided into two times of day.

- Safe use:

- Before filling, make sure the compartments are clean, dry, and properly closed.

- Fill the organizer only with medicines taken according to the instructions of a doctor, pharmacist, or the patient leaflet.

- Before taking any medicine, always check the correct day and time of day to avoid dosing mistakes.

- Warnings:

- This product is not a toy. Keep it out of sight and reach of children and animals.

- The organizer is not child-resistant and does not replace the original medicine packaging, label, or leaflet.

- Do not use the product if it is cracked, deformed, damaged, or does not close properly.

- Do not store medicines that require refrigeration, special protection from light or moisture, or medicines that should remain in their original packaging.

- Do not use the organizer for liquids, powders, ointments, or other substances that may leak or contaminate the compartments.

- Cleaning and maintenance:

- Before first use and before each subsequent use, wash the organizer by hand with warm water and mild detergent, then rinse thoroughly and dry completely.

- Do not clean the product with aggressive chemicals or sharp tools.
 - Storage:
 - Store the product in a dry and clean place, away from heat sources and direct sunlight.
 - Additional information:
 - If you have any doubts about repackaging or combining medicines, consult a doctor or pharmacist.
 - In the event of a dosing mistake or suspected accidental ingestion by a child, contact a doctor or poison control center immediately.
-

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